




Specialised
Orthotic
Services

Units 127/128
Fauld Industrial Park
Tutbury, Nr Burton upon Trent, Staffordshire.
DE13 9HS
Tel: 0044 (0)1283 520 400
Fax: 0044 (0)1283 520 401
E-mail: enquiries@specialorthotic.com
Web: www.specialisedorthoticservices.co.uk

MOULDED SANDALS FITTING INSTRUCTIONS

Moulded sandals are designed to be attached to most wheelchair footrests. Most standard fastenings, such as threaded fastenings and pop rivets are more than adequate to secure the moulded sandal.

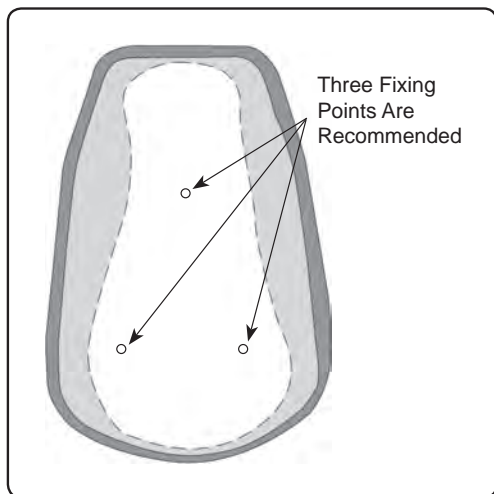

Fasteners less than 5mm in body size should not be used, as these will be too weak to secure the sandal safely.

When attaching the sandal, always carry out any drilling of the footplate away from the client.

It is always a good idea, whenever possible, to fit the sandal with the client present. This will allow you to accurately place the feet and make allowances for any abnormalities that may need accommodation.

Make sure there is some free space inside the sandal when the clients shoe is located.
NEVER FORCE THE CLIENTS FEET INTO THE SANDAL.

CLEANING

The sandals can be easily cleaned, by wiping with a damp cloth.

PLEASE NOTE: WHEN USING THE SANDALS THE CLIENT MUST ALWAYS BE WEARING FOOTWEAR (SHOES). SLIPPERS ARE NOT RECOMMENDED.

