



Specialised
Orthotic
Services

POZIFORM[®]

CALF PANEL

*Installation &
User Instructions*

POZIFORM[®] is a registered trademark of Specialised Orthotic Services Ltd.

Thank you for purchasing your **POZIFORM[®] CALF PANEL**.

The **POZIFORM[®]** Calf Panel is designed to provide a lightweight, padded support to prevent the wheelchair users feet from sliding off the foot rests underneath the wheelchair.

All **POZIFORM[®]** products are supplied with a comprehensive 3 months warranty from the date of purchase (For terms and conditions please consult our Customer Services Department).

To fit your **POZIFORM[®]** Calf Panel please follow these instructions supplied.

Specialised Orthotic Services Ltd

Units 127/128 Fauld Ind Park

Tutbury, Nr Burton-on-Trent,

Staffordshire. ENGLAND. DE13 9HS

Tel: 0044 (0)1283 520 400 Fax: 0044 (0)1283 520 401

E-mail: enquiries@specialorthotic.com

Web: specialisedorthoticservices.co.uk



Calf Panel should look like this when fitted to wheelchair.

KIT LIST & TOOL REQUIREMENTS

Your Harness Kit Consists Of The Following:



1 x Calf Panel

Tools Required (not supplied):

NO TOOLS REQUIRED

CLEANING & MAINTENANCE

Cleaning Instructions:

Clean down with a damp cloth and a mild detergent.
When wet allow to dry naturally in a warm dry atmosphere.

DO NOT dry in any drying machine.

DO NOT wash in an automatic washing machine.

DO NOT iron.


Maintenance Instructions:

CHECK for any signs of fraying and that straps work correctly.


PERIODIC cleaning will help prevent damage to straps.

FITTING INSTRUCTIONS

BEFORE fitting the harness to any wheelchair/seating system please consult the manufacturers hand book (or contact them direct) to find out if it is safe to do so.

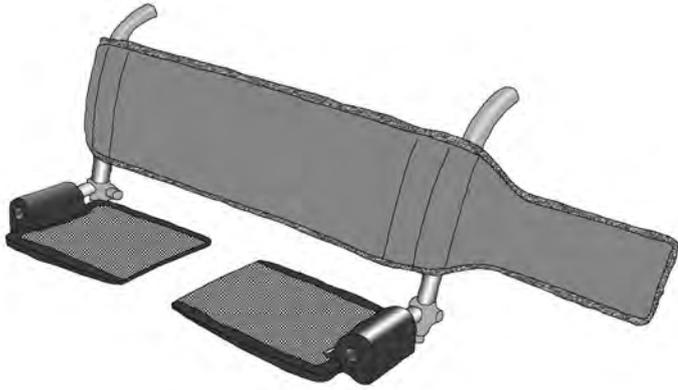

- 1) Wrap Calf Panel around the front of the foot rest stems as shown.

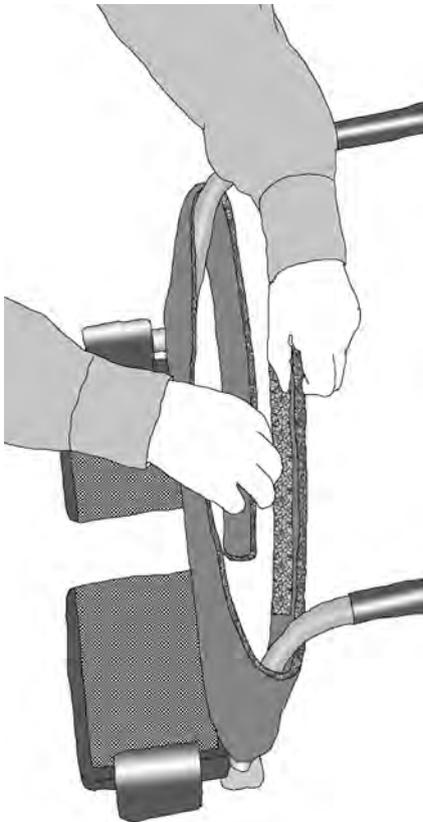

- 2) Pull the Calf Panel tight and press the two sides of the velcro together.

Note: You must remove the Calf Panel before removing or swinging the foot stems aside to prevent damage to Calf Panel and wheelchair. Removal of the Calf Panel is a reversal of the fitting procedure.