



Specialised  
Orthotic  
Services

# Stroller 2

## Sunshade & Rain Canopy



# USER'S MANUAL

ALL USER'S OF THE EQUIPMENT SHOULD BE AWARE  
OF THIS DOCUMENT AND ITS CONTENT

## **CONTENTS** *Kit includes either type 1 or 2*

### **TYPE 1: COMBI SUNSHADE/RAIN CANOPY**



- 1 x Assembled Sunshade (c/w fixings)*
- 1 x Rain Canopy*

### **TYPE 2: SOLO RAIN CANOPY**



- 1 x Assembled Rain Canopy  
(c/w fixings)*

# TYPE 1: COMBI SUNSHADE/RAIN CANOPY

## 1.1. STROLLER 2 COMBI SUNSHADE FITTING INSTRUCTIONS

- Fix the two frame clips on to the handle frame just below the pivot joint (Fig.1.1a). Repeat for other side.
- Push the centre of the frame locking bar until straight so that the sunshade is tensioned properly (Fig.1.1b). Repeat for other side.
- Correctly fitted & tensioned sunshade as shown (Fig.1.1c)
- Removal is a reversal of the above.



## 1.2. STROLLER 2 COMBI RAIN CANOPY FITTING INSTRUCTIONS


- Remove the rain canopy from the zipped pouch on the back of the sunshade (Fig.1.2a).
- Unfold the rain canopy and hook over the sunshade as shown (Fig.1.2b).
- Pull the rear panel down between the Stroller handles and hook the front of the rain canopy around the footbox on the seating system (if fitted).
- Hook the two elasticated straps around the rear tie down hooks as shown (Fig.1.2c). Repeat for other side.
- Correctly fitted rain canopy as shown (Fig.1.2d)
- Removal is a reversal of the above.
- **Please ensure the top & bottom retaining straps (Fig.1.2c) are released BEFORE folding/removing to prevent damage to the rain canopy.**



# TYPE 2: SOLO RAIN CANOPY

## 2.1. STROLLER 2 SOLO RAIN CANOPY FITTING INSTRUCTIONS

- Fix the two frame clips on to the handle frame just below the pivot joint (Fig.2.1a). Repeat for other side.
- Unfold the rain canopy and fix the two halves of the Velcro straps (at top & bottom) between the two clear panels (Fig.2.1b). Repeat for other side.
- Correctly fitted rain canopy as shown (Fig.2.1c)
- Removal is a reversal of the above.

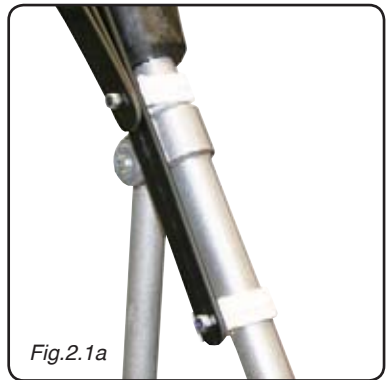

Should any problems arise due to faulty manufacturing & materials under normal everyday use then it is covered by our comprehensive 12 months warrantee from the date of purchase (For terms and conditions please consult our Customer Services Department).

***Specialised Orthotic Services Ltd*** cannot be held responsible for any unauthorised use or incorrect fitting whether resulting in personal injury or not and any damage caused to equipment not supplied with the tray.

The company has a policy of continual development and we reserve the right to alter specification and design without prior notification. All products manufactured are covered by design rights or patents. This document is the copyright of the company and the contents are its intellectual property.

## NOTES

If you require further copies of this handout or require further details relating to any of its content, then please contact us (Copyright applies):



**Unit 127/128, Fauld Industrial Park, Tutbury, Staffordshire. DE13 9HS.  
Tel: 01283 520400 Fax: 01283 520401  
E-mail: [enquiries@specialorthotic.com](mailto:enquiries@specialorthotic.com)**