



Specialised
Orthotic
Services

Units 127/128
Fauld Industrial Park
Tutbury, Nr Burton upon Trent, Staffordshire.
DE13 9HS
Tel: 0044 (0)1283 520 400
Fax: 0044 (0)1283 520 401
E-mail: enquiries@specialorthotic.com
Web: www.specialisedorthoticservices.co.uk

3" ARMREST RAISER FITTING INSTRUCTIONS



CONTENTS

You should have the following:

1. 3" Front Raiser (x1)
2. 3" Rear Raiser (x1)
3. M6 x 8mm Grub Screw (x2)
4. M8 x 25mm Washer (x1)
5. M8 x 25mm Screw (x1)

If any of the above items are missing, please contact our Customer Services Department on 0044 (0)1283 520400.

1. FITTING

- Remove the original armrest from the wheelchair (Fig.1a) and place on a suitable work surface.



Fig.1a

- Slide the Front Raiser on to the armrest and tighten in place until the M6 grub screw is flush (Fig.1b), the grub screw will go into the original hole on the armrest for location.
- Slide the Rear Raiser on to the armrest and tighten in place until the M6 grub screw is flush (Fig.1c), the grub screw will go into the original hole on the armrest for location.



- Refit the armrest and Raisers to the wheelchair, making sure the Front Raiser is held in place by the armrest locking lever (Fig.1d) and the Rear Raiser locates in the rear socket (Fig.1e).
- Once the armrest is in place, fix the Rear Raiser in place with M8x25 Screw & M8 Washer provided (Fig.1e) and tighten in place.
- If the armrest needs to be frequently removed (i.e side transfers, transport, etc..) then remove the grub screw from the Rear Raiser (Fig.1c). Removal of armrest is via the release lever on the wheelchair and removal of grub screw in Rear Raiser (if fitted).

