



Specialised
Orthotic
Services

Units 127/128
Fauld Industrial Park
Tutbury, Nr Burton upon Trent, Staffordshire.
DE13 9HS
Tel: 044 (0)1283 520 400
Fax: 044 (0)1283 520 401
E-mail: enquiries@specialorthotic.com
Web: www.specialisedorthoticservices.co.uk

ARMREST OUTRIGGER FITTING INSTRUCTIONS



CONTENTS

You should have the following:

1. Rear Armrest Outrigger (x2)
2. Front Armrest Outrigger (x2)
3. Front Spigot (x2)
4. M8 x 16mm Screw (x2)
5. M8 Washer (x2)
6. M8 x 25mm Screw (x2)
7. M8 x 25mm Washer (x2)
8. M6 x 8mm Grub Screw (x4)

If any of the above items are missing, please contact our Customer Services Department on 044 (0)1283 520400.

1. FITTING

- Remove the original armrest from the wheelchair (Fig.1a).
- Fit the Rear Outrigger into the armrest rear slot on the wheelchair and secure in place with M8x25 Screw & M8x25 Washer provided (Fig.1b). Do Not fully tighten just yet.

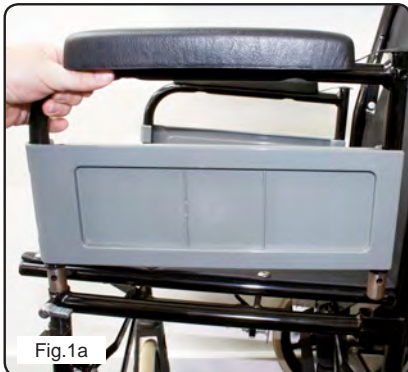


Fig.1a


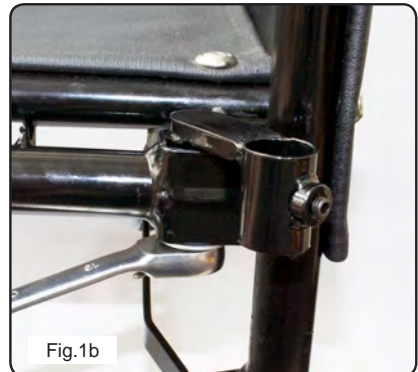


Fig.1b

- Fit the Front Outrigger into the front armrest socket (Fig.1c), making sure the Front Outrigger is held in place by the armrest locking lever.
- Fit the armrest into the Front (Fig.1d) and Rear (Fig.1e) Outriggers, rotate the Outriggers until position required and tighten the grub screws in place (Fig.1d & e).
- Fully tighten the screw in the Front Outrigger (Fig.1f) and the screw in the Rear Outrigger (Fig.1b).



- To remove the armrest, undo the two grub screws and lift off the armrest.

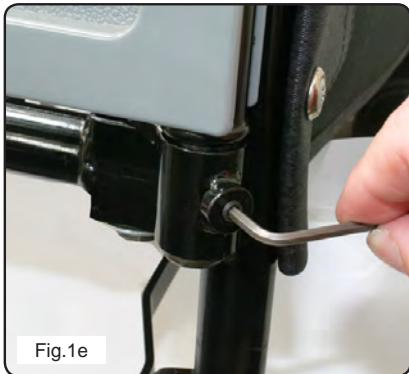