



Specialised
Orthotic
Services

Units 127/128
Fauld Industrial Park
Tutbury, Nr Burton upon Trent, Staffordshire.
ENGLAND. DE13 9HS
Tel: 0044 (0)1283 520 400
Fax: 0044 (0)1283 520 401
E-mail: enquiries@specialorthotic.com
Web: www.specialisedorthoticservices.co.uk

40mm FRAME CLAMP FITTING INSTRUCTIONS



CONTENTS

You should have the following:

1. 3/4" Frame Adapter (x2)
2. 7/8" Frame Adapter (x2)
3. 1" Clamp (x2)
4. M6 Washer (x2)
5. M6 x 35mm Screws (x2)
6. 40mm Webbing Guide Plate (x2)

If any of the above items are missing, please contact our Customer Services Department on 0044 (0)1283 520400.

1. FITTING

- Clip the appropriate size Frame Adapter if required (Horizontal fitting Fig.1a or Vertical fitting Fig.1b) around the frame tube, as close to the rear of the seat base as possible.

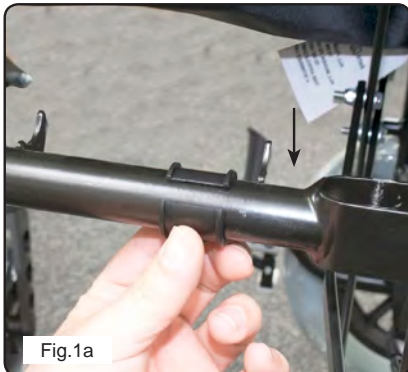


Fig.1a



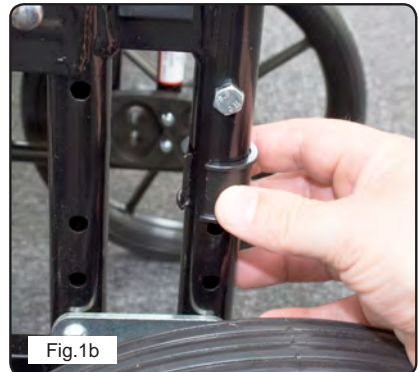


Fig.1b

- Clip the Frame clamp around the frame tube or the frame adapter if fitted (Horizontal fitting Fig.1c or Vertical fitting Fig.1d) making sure the threaded insert is facing towards the wheelchair.

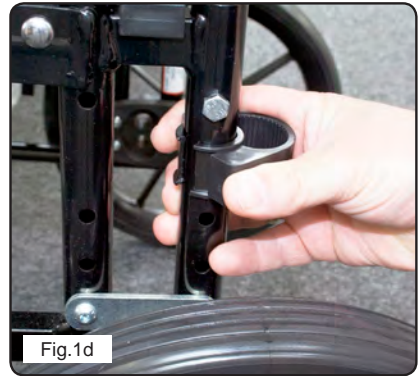

- Mount the webbing guide plate to the frame clamp with the M6 washers and M6 screws supplied (Horizontal fitting Fig.1c or Vertical fitting Fig.1d). Position the webbing guide plate in the orientation as required and ensure the screws are fully tightened.

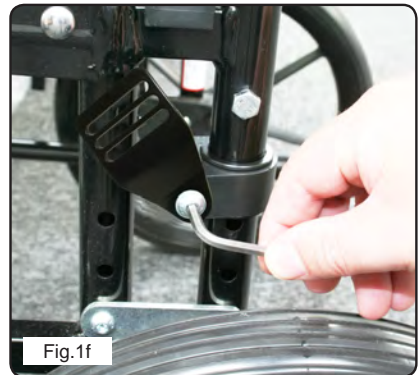

- Attach the harness webbing (not supplied) to the slots on the webbing guide plate.

