



Specialised  
Orthotic  
Services

**POZIFORM<sup>®</sup>**

**COMFORT PAD**

*Installation &  
User Instructions*

**POZIFORM<sup>®</sup>** is a registered trademark of Specialised Orthotic Services Ltd.

Thank you for purchasing your **POZIFORM<sup>®</sup> COMFORT PAD.**

Comfort Pads are a convenient way to provide additional support and comfort to client belt, harnesses, straps, etc.

Available in a range of styles, comfort pads are easily fitted to most standard types of webbing.

All **POZIFORM<sup>®</sup>** products are supplied with a comprehensive 3 months warranty from the date of purchase (For terms and conditions please consult our Customer Services Department).

To fit your **POZIFORM<sup>®</sup>** Comfort Pad please follow these instructions supplied.

## Specialised Orthotic Services Ltd

Units 127/128 Fauld Ind Park

Tutbury, Nr Burton-on-Trent,

Staffordshire. ENGLAND. DE13 9HS

Tel: 0044 (0)1283 520 400 Fax: 0044 (0)1283 520 401

E-mail: [enquiries@specialorthotic.com](mailto:enquiries@specialorthotic.com)

Web: [specialisedorthoticservices.co.uk](http://specialisedorthoticservices.co.uk)



Comfort Pad Range

### KIT LIST & TOOL REQUIREMENTS

#### Comfort Pad Types:



1 x Fitting Instructions  
1 x Standard Comfort Pad

STANDARD: Sizes 1 to 2

OR



1 x Fitting Instructions  
1 x Tapered Comfort Pad

TAPERED: Sizes 1 to 4

OR



1 x Fitting Instructions  
1 x Maxi Comfort Pad

MAXI: Sizes 2 to 3

#### Tools Required (not supplied):

NO TOOLS REQUIRED.

## CLEANING & MAINTENANCE

### Cleaning Instructions:


Clean down with a damp cloth and a mild detergent.  
When wet allow to dry naturally in a warm dry atmosphere.  
**DO NOT** dry in any drying machine.  
**DO NOT** wash in an automatic washing machine.  
**DO NOT** iron.

### Maintenance Instructions:

**CHECK** for any signs of fraying.  
**PERIODIC** cleaning will help prevent damage to pad.

## FITTING INSTRUCTIONS

- 1) Make sure you are fitting the comfort pad to the correct width webbing strap.

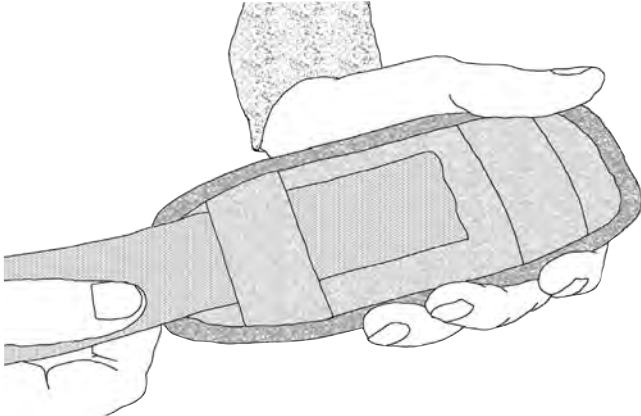

- 2) Slide strap under the two fixed loops as shown.
- 3) Slide comfort pad into required position.

### Examples of Comfort Pad Usage



Standard Comfort Pads used in conjunction with POZIFORM® Foot Straps.



Standard Comfort Pad used as a shoulder pad on a holdall shoulder strap.