




Specialised
Orthotic
Services

Units 127/128
Fauld Industrial Park
Tutbury, Nr Burton upon Trent, Staffordshire.
ENGLAND. DE13 9HS
Tel: 0044 (0)1283 520 400
Fax: 0044 (0)1283 520 401
E-mail: enquiries@specialorthotic.com
Web: www.specialisedorthoticservices.co.uk

FOOT STRAP FITTING INSTRUCTIONS

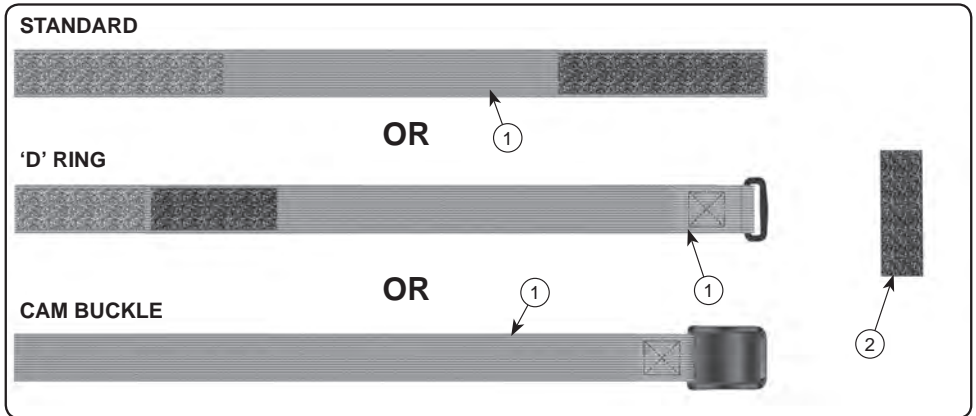

CONTENTS

You should have the following:

1. Foot Strap (2 off)
2. Self Adhesive Velcro Pad (2 off)



If any of the above items are missing, please contact our Customer Services Department on 0044 (0)1283 520400.

Foot Straps are supplied in pairs that can be used on one foot (as shown) or individually on two feet.

BEFORE fitting the Foot Straps to any wheelchair/seating system please consult the manufacturers hand book (or contact them direct) to find out if it is safe to do so.

1. FOOT STRAP FITTING

- Place users foot (or shoe) on footbox/plate and mark out two slots per strap (Fig.1a).
- Pull the strap through the outer slot (Fig.1a), making sure the underside velcro is facing upwards.
- Attach the self adhesive velcro on the underside of footbox/plate (Fig.1b) between the two slots.
- If you have foot plates with slots already in them, please utilize these (Fig.1c). Fitting is the same as before.
- Alternatively (Fig.1d) you can drill through the straps and attach with screws/rivets etc (not supplied).

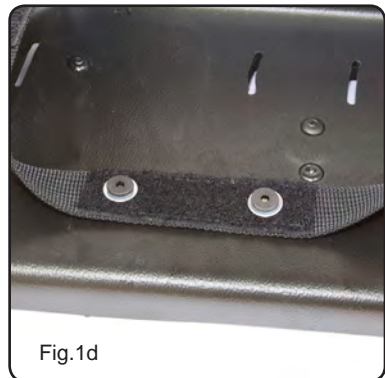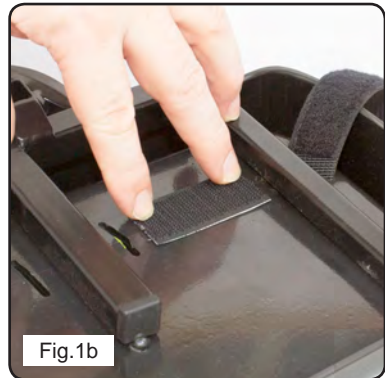

2. HARNESS ADJUSTING

- Please use the following instructions which are applicable to your type of strap:

STANDARD

- Grasp both halves of the strap, pull them until tensioned (Fig.2a) and press the two halves of the velcro together.

'D' RING

- Hold the 'D' ring, pull the strap through until tensioned (Fig.2b) and press the two halves of the velcro together.

CAM BUCKLED

- Open the flap on the cam buckle and pull the strap through until tensioned (Fig.2c), once tensioned close the flap on the cam buckle.

Removal is a reverse of the above procedures.



3. CLEANING INSTRUCTIONS

- Clean down with a damp cloth and a mild detergent.
- When wet allow to dry naturally in a warm dry atmosphere.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in an automatic washing machine.
- **DO NOT** iron.

4. MAINTENANCE INSTRUCTIONS

- **CHECK** for any signs of fraying and that buckles and straps work correctly.
- **CHECK** and make sure that all fixings are secure and not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to buckle and straps.

