




Specialised
Orthotic
Services

Units 127/128
Fauld Industrial Park
Tutbury, Nr Burton upon Trent, Staffordshire.
ENGLAND. DE13 9HS
Tel: 0044 (0)1283 520 400
Fax: 0044 (0)1283 520 401
E-mail: enquiries@specialorthotic.com
Web: www.specialisedorthoticservices.co.uk

HARNESS FIXING KIT FITTING INSTRUCTIONS

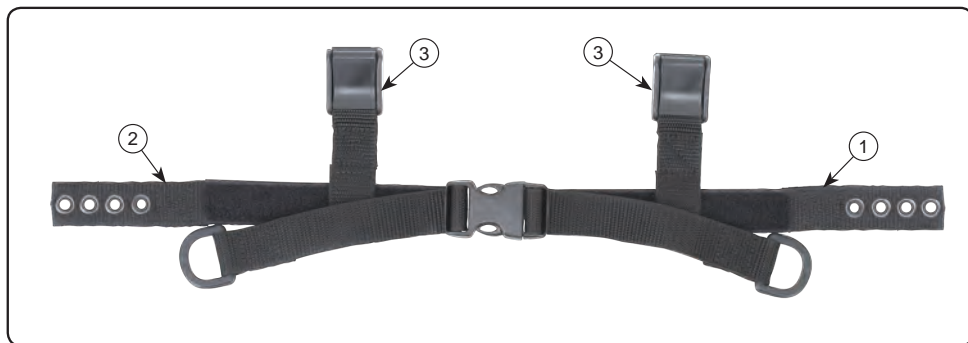

CONTENTS

You should have the following:

1. Male Strap (x1)
2. Female Strap (x1)
3. Harness Retaining Cam Buckles (x2)
4. 3/4" Frame Adapter (x2)
5. 7/8" Frame Adapter (x2)
6. 1" Clamp (x2)
7. M6 Washer (x2)
8. M6 x 35mm Screws (x2)
9. 25mm Webbing Guide Plate (x2)

If any of the above items are missing, please contact our Customer Services Department on 01283 520400.

BEFORE fitting the harness to any wheelchair/seating system please consult the manufacturers hand book (or contact them direct) to find out if it is safe to do so.

1. STRAP FITTING

- Remove two Canvass Screws (if fitted), Locate Screws & Cup Washers through the eyelets in the Straps (Fig.1a) and attach to Wheelchair rear canvass up-rights (holes may be hidden under canvass) Note: Screws may be obtained from your local wheelchair clinic.
- Attach the two straps together by grasping the two halves of the buckle (Fig.1b) and securely pressing together.
- Grasp the two 'D' rings (Fig.1c) and pull the webbing through the buckle until tensioned.
- The Harness Fitting Kit does not have to be removed (or undone) when folding the wheelchair, but if the need arises to remove the kit then release the buckle via the side buttons (Fig.1d) and remove as a reversal of the above.



Fig.1a



Fig.1b



Fig.1c



Fig.1d

2. TUBE CLAMP FITTING

- Clip the appropriate size Frame Adapter if required (Horizontal fitting Fig.2a or Vertical fitting Fig.2b) around the frame tube, as close to the rear of the seat base as possible.
- Clip the Frame clamp around the frame tube or the frame adapter if fitted (Horizontal fitting Fig.2c or Vertical fitting Fig.2d) making sure the threaded insert is facing towards the wheelchair.
- Mount the webbing guide plate to the frame clamp with the M6 washers and M6 screws supplied (Horizontal fitting Fig.2e or Vertical fitting Fig.2f). Position the webbing guide plate in the orientation as required and ensure the screws are fully tightened.

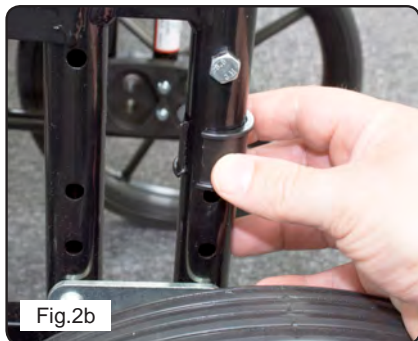




Fig.2e

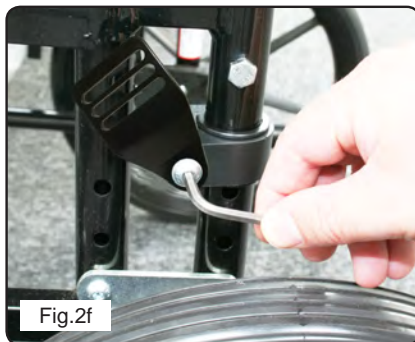


Fig.2f

- Attach the harness webbing (not supplied) to the slots on the webbing guide plate.

3. CLEANING INSTRUCTIONS

- Clean down with a damp cloth and a mild detergent.
- When wet allow to dry naturally in a warm dry atmosphere.
- **DO NOT** dry in any drying machine.
- **DO NOT** wash in an automatic washing machine.
- **DO NOT** iron.

4. MAINTENANCE INSTRUCTIONS

- **CHECK** for any signs of fraying and that buckles work correctly.
- **CHECK** & make sure that all fixings are secure & not damaged, especially after making any adjustments.
- **PERIODIC** cleaning will help prevent damage to buckle.

