




Specialised  
Orthotic  
Services

Units 127/128  
Fauld Industrial Park  
Tutbury, Nr Burton upon Trent, Staffordshire.  
DE13 9HS  
Tel: 01283 520 400  
Fax: 01283 520 401  
E-mail: [enquiries@specialorthotic.com](mailto:enquiries@specialorthotic.com)  
Web: [www.specialisedorthoticservices.co.uk](http://www.specialisedorthoticservices.co.uk)

## BABY JOGGER TRANSPORTATION KIT FITTING INSTRUCTIONS

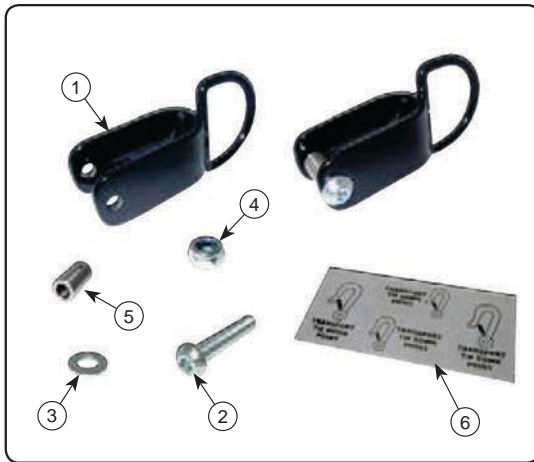

### CONTENTS

You should have the following:

1. Rear Tie Down Bracket (x 2)
2. M6 x 35mm Screws (x 2)
3. M6 Washer (x 2)
4. M6 Nyloc Nut (x 2)
5. Spacer (x 2)
6. Tie Down Labels (set of 4)

If any of the above items are missing, please contact our Customer Services Department on 01283 520400.

### 1. FITTING

- Slide the Tie Down Bracket (Fig.1a) around the LH frame tube, just below the buggy storage frame lock.
- Fix the Bracket in place using the fittings supplied (Fig.1b) making sure there is a Washer under the head of the Screw and the Nyloc Nut, and that the Spacer is used to prevent distortion when tightening .



- Apply the Tie Down label on to the Tie Down Bracket (Fig.1c) making sure the picture of the hook is closest to the 'D' ring. Repeat the above fitting for the RH Bracket, setting it at the same height as the LH Bracket.
- Apply the remaining two Tie Down Labels to the front cross beam as shown (Fig.1d)



## 2. TRANSPORTATION

- Apply the rear brakes. **Please Note:** The Stroller & seating system can be safely transported at any angle of tilt as long as the seat belt can be located correctly over the shoulder.
- Attach the W.T.O.R.S. to the front cross beam (Fig.2a).
- Attach the W.T.O.R.S. to the rear tie down points (Fig.2b).
- Attach a permanently fixed vehicle seat belt around the client and wheelchair.
- **Please refer to the W.T.O.R.S. manufacturers handbook for correct fitting and adjustment for both the client and passengers safety.**
- **Please refer to user manual supplied with buggy for more detailed information on transportation.**

